

# Cup & Cane Holders

For Wheelchairs, Walkers, and Rollators  
Tool-Free Assembly!

## Cup Holder - 3 Easy Steps!

1. Make sure you have all the necessary pieces.



U-Shaped Clamp



Two (2) Scored Rubber Inserts



Cup Holder with Star Knob


2. Snap u-shaped clamp onto the tubing of your walker, rollator, or transport chair. Be sure the clamp is positioned in such a way that allows the cup to sit level, NOT tilted in a direction that might allow your drink to spill.



NOTE: If necessary, insert the sleeve in the clamp to better fit the size of the tubing. Whether you use the sleeve or not, the clamp should fit tightly on the tubing, otherwise the holder is susceptible to slipping and spilling your drink.



3. Remove the star-shaped knob from the cup holder, insert the holder through the opening in the clamp, and tighten the knob to assure that the cup holder stays tightly attached.

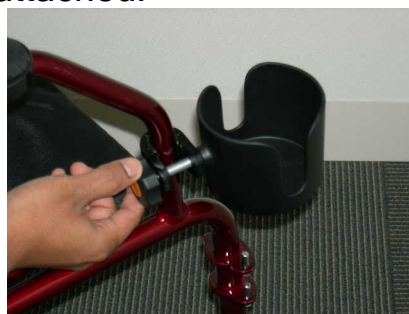

Enjoy your drink on the go!



# Cane Holder - 5 Easy Steps!

1. Make sure you have all the necessary pieces.



Rotating Basket



Rotating Two-Hole Clamp



Six (6) Plastic Inserts



Velcro Strap

2. Affix the rotating basket toward the bottom of your mobility device by removing the star-shaped knob, fitting the opening around the tubing on the device, and screwing in the knob tightly. The basket should face to the outside as it serves as a resting place for the base of your cane.



NOTE: If necessary, use one of the insert pieces to better fit the tubing of your mobility device. The basket also rotates and locks into 4 different positions depending on the angle your cane is best suited to rest.



3. Affix the rotating two-hole clamp toward the top of your mobility device, keeping in mind the base of the cane will sit in the basket and this piece actually locks the cane into place. Remove the star-shaped knob, fit the opening around the tubing on the device, and screw in the knob tightly. The open-faced clamp should face to the outside, 1-2 feet above the basket.



NOTE: If necessary, use one of the insert pieces to better fit the tubing of your mobility device. The clamp also rotates and locks into 4 different positions depending on the angle your cane is best suited to rest.






4. Snap the cane into the upper piece and allow the base to rest in the basket. Adjust basket/piece placement so that the cane is balanced and secure.



5. For added security, snap the velcro piece around the tubing of the mobility device and wrap the strap around the cane.

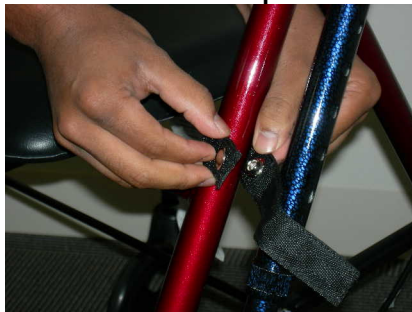

Now, your cane is ready to go while you're on the go!





## **WARNING**

**TO ENSURE YOUR SAFETY IN USING THESE MEDLINE CUP AND CANE HOLDERS, ALL WARNINGS AND SAFETY INFORMATION AND ALL INSTRUCTIONS MUST BE FOLLOWED. FAILURE TO DO SO MAY RESULT IN SERIOUS BODILY INJURY OR DAMAGE TO THE PRODUCT.**

**DO NOT USE THESE PRODUCTS WITHOUT FIRST READING AND UNDERSTANDING THE INSTRUCTIONS CONTAINED IN THIS BOOKLET. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DURABLE MEDICAL DEALER OR TECHNICAL PERSONNEL, OR CONTACT MEDLINE AT 1-800- MEDLINE BEFORE USE – OTHERWISE SERIOUS BODILY INJURY OR PRODUCT DAMAGE MAY OCCUR.**

**A QUALIFIED TECHNICIAN MUST PERFORM ANY REPAIR OR CHANGE TO THESE PRODUCTS OR SERIOUS INJURY MAY OCCUR. TECHNICIANS: DO NOT SERVICE THESE PRODUCTS WITHOUT FIRST READING AND UNDERSTANDING THIS OWNER'S MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, OR HAVE ANY QUESTIONS, CONTACT MEDLINE AT 1-800- MEDLINE.**

**DO NOT USE ACCESSORIES, PARTS, OR ADAPTERS OTHER THAN THOSE AUTHORIZED BY MEDLINE.**

**CONSULT WITH YOUR DOCTOR, PHYSICAL OR OCCUPATIONAL THERAPIST, OR OTHER QUALIFIED PROFESSIONAL REGARDING THE SELECTION OF THE TYPE OF ACCESSORIES BEST SUITED TO YOUR INDIVIDUAL NEEDS AND CAPABILITIES.**

**SAVE THESE INSTRUCTIONS FOR FUTURE  
REFERENCE!**

---

### **Lifetime Limited Warranty**

Your Medline brand product is warranted to be free of defects in material and workmanship for a lifetime for the original consumer.

This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components which are subject to normal wear and replacement. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you many have other rights, which vary from state to state.

**This Product Contains Dry Natural Rubber.**