

 **LifeEar**
Read before use.
v.A1215


 **LifeEar**




Simple Device. Enhanced Life.

Box Contents

LifeEar with tubing
& tip attached.

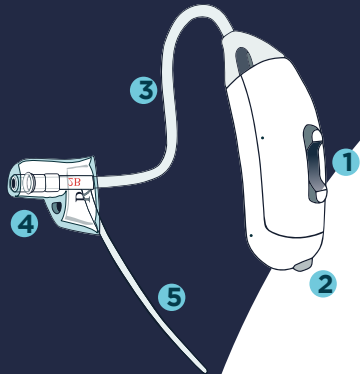

Size 10 Batteries
(Yellow Tab)

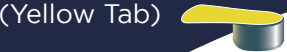

Power Tip
Large



Cleaning
Brush & Wire



- 1 Control Switch
- 2 Battery Door
- 3 Tubing
- 4 Standard LifeEar Tip
- 5 Ear Lock

Power:

ON = battery door closed
OFF = battery door open

Battery Type:


LifeEar uses a size 10 zinc-air battery (yellow tab).

*Wait one FULL minute
after removing tab for
battery to fully charge.*

Low Battery Warning:

Three rapid double beeps


Inserting Battery

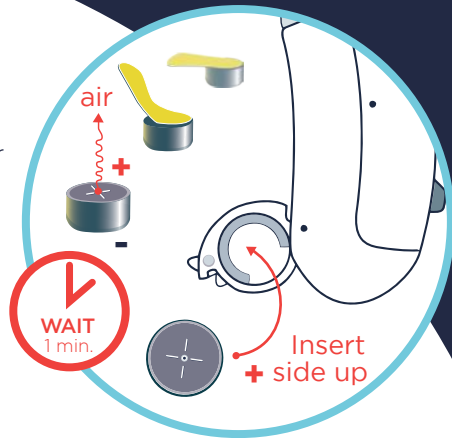


Wearing Your LifeEar

Place tip in entrance of ear canal.

1

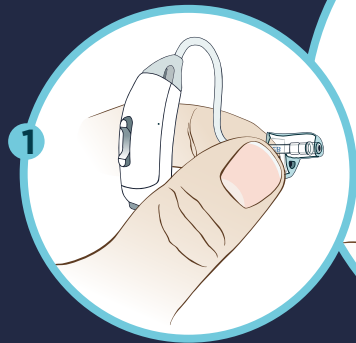


2

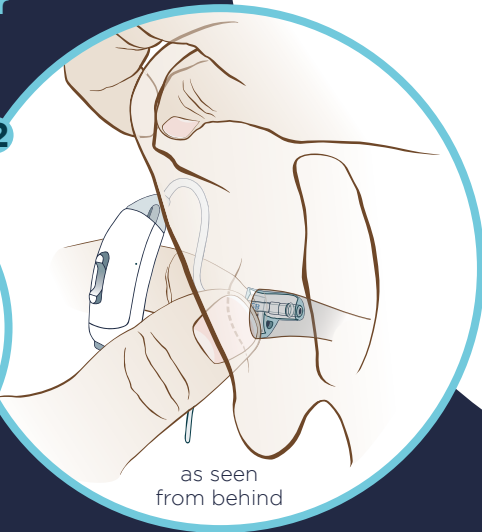

as seen
from behind


3

Place LifeEar behind ear.

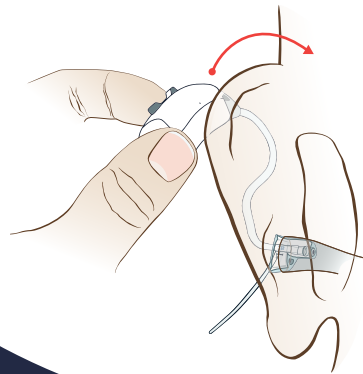


4

Move ear lock into bowl of ear.

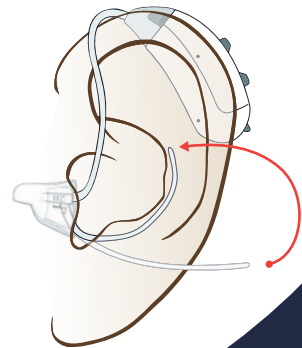

Controlling Program & Volume

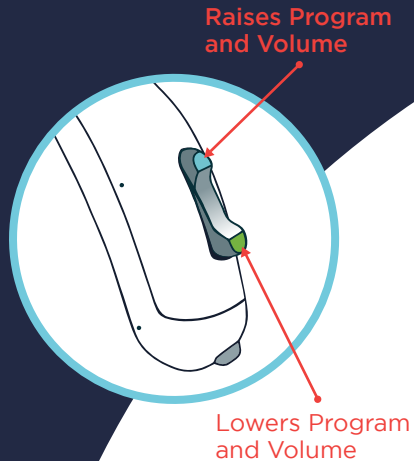

Volume: Press/Release before LifeEar beeps. Single beep when volume adjusted. Triple beeps at volume limit.

Program: Press/Hold until LifeEar beeps.

- beep ♪ = Program 1
- beep, beep ♪ ♪ = Program 2
- ♪ ♪ ♪ = Program 3
- ♪ ♪ ♪ ♪ = Program 4

Adjusting Volume:

Press and Release switch until normal conversation is comfortable.



Set volume in quiet room.

Program Settings:

- 1 Enhanced sound across all frequencies.
- 2 High-frequencies emphasized.
- 3 Program 1 plus noise reduction boost.
- 4 Program 2 plus noise reduction boost.

Replacing LifeEar Tubing & Tips







Tubing: Rotate base 90° and slide off body of LifeEar. Push new tubing on.

Tip: Pull tip off tubing. Push new tip on until flush with end of tubing.



Replace every three months for best sound.
www.lifeear.com

Fitting Your LifeEar


	Average Ear Canal	Large Ear Canal
Natural Sound 	included  LifeEar Tip Standard	 * LifeEar Tip Large
Maximum Output 	 * Power Tip Standard	included  Power Tip Large

*Available at www.lifeear.com

Cleaning & Care

Avoid exposure to moisture.

Clean tubing regularly with enclosed tool.

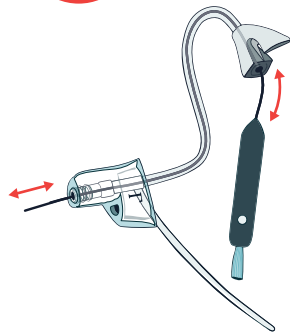

Support

www.lifear.com

Troubleshooting

Problems? Issues? Questions?
Visit www.lifear.com

Note: If your LifeEar provides no amplification with tubing attached, clean tubing thoroughly.