Lansinoh®

MANUAL BREAST PUMP
INSTRUCTIONS
FOR USE

Introducing the Lansinoh® Manual Breast Pump

The Lansinoh® Manual Breast Pump has been designed specifically with the needs of breastfeeding mothers in mind. This breast pump has two modes: Let-down Mode and Expression Mode. You can easily switch between the two modes by adjusting the position of the handle on the stem. The ComfortFit™ flange creates a seal providing reliable suction resulting in comfortable and fast milk flow (standard and large size included). The wide neck bottle is interchangeable with the entire Lansinoh® pump range. The pump is easy to assemble, easy to use and easy to clean.

The American Academy of Pediatrics and other major medical organizations recommend that babies be exclusively breastfed for 6 months, breastfed for 12 months with complementary foods and thereafter for as long as the mother and baby wish to continue. To assist you in meeting these goals, Lansinoh® has created this Manual Breast Pump to help you provide breastmilk for your baby while you are separated, to help keep up your supply and to have your breastmilk available for him or her.

Important Safeguards

The Manual Breast Pump is a personal care item and for hygienic reasons should never be shared between mothers or resold from one mother to another. If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breastmilk using the Lansinoh Manual Pump will not reduce or remove the risk of transmitting the virus to your baby through your breastmilk.

**Save These Instructions
along with your receipt for future reference.**

Congratulations!

Your decision to provide your baby with breastmilk, even if you need to be apart, is one of the most precious gifts you can give your baby. Lansinoh's Manual Pump is a safe and effective choice for moms who need to pump occasionally and who appreciate the convenience of a manual pump.

Contents	Page
1 Lansinoh® Manual Breast Pump Parts.....	4
2 How to Clean and Sterilize Your Lansinoh® Manual Breast Pump.....	5
3 How to Assemble Your Lansinoh® Manual Breast Pump.....	6
4 How to Use Your Lansinoh® Manual Breast Pump.....	8
5 Breastmilk Storage Guidelines.....	11
6 Tips About Pumping.....	12
7 Troubleshooting for the Lansinoh® Manual Breast Pump.....	13

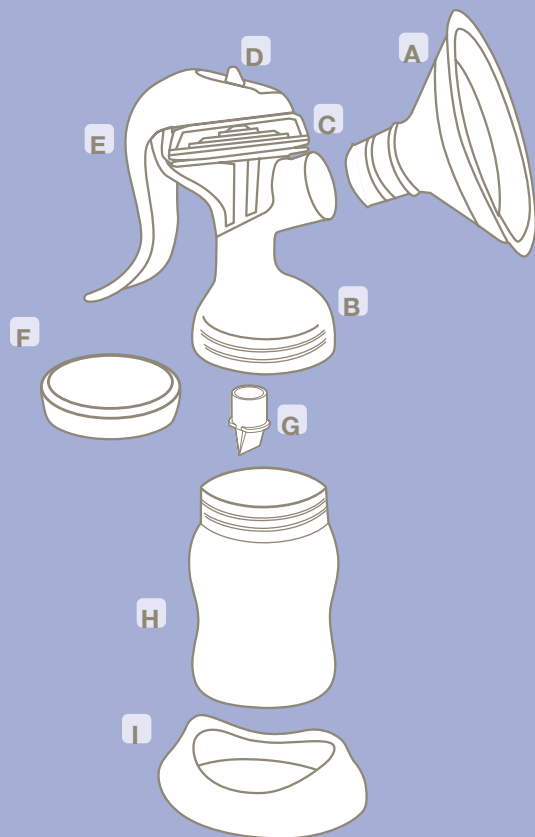
PLEASE DO NOT RETURN THIS PUMP TO THE RETAILER.

Lansinoh's dedicated Customer Service team is here to help. If you experience any issues with your Lansinoh® Manual Pump please call 800-292-4794 or email customerservice@lansinoh.com, Monday - Friday 8am-8pm EST.

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H Collection Bottle

I Bottle Stand

2. HOW TO CLEAN AND STERILIZE YOUR LANSINOH® MANUAL BREAST PUMP

Sterilize:

Prior to using your pump for the first time, disassemble it (remove silicone diaphragm and stem, ComfortFit™ flange and white valve) and boil all parts, including bottle stand, for 10 minutes. Remove all parts from water immediately after boiling. Completely air dry before assembly and before the next use.

NOTE: It is essential that you use enough water to keep the parts freely floating while boiling.

Cleaning:

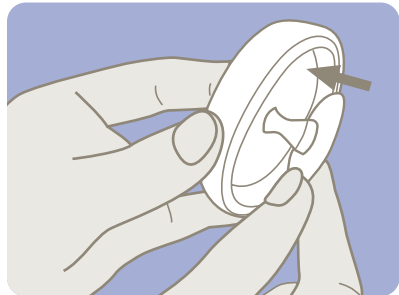
The breast flange, bottle, handle and bottle stand may be cleaned in the upper compartment of your dishwasher or hand washed with dish soap and hot water. Do not use solvents or abrasives. Rinse with hot, clean water. The white valve, silicone diaphragm, stem, and the ComfortFit™ flange can be washed in the dishwasher; however, we recommend washing them by hand to prevent loss and to extend their usefulness. Completely air-dry all parts on a clean towel or drying rack.

NOTE: Use caution when cleaning the small white valve. It is important for proper suction. Do not use excessive force and do not clean with a baby bottle brush or a nipple brush. Gently wash by hand in hot, soapy water and rinse under hot running water.


3. HOW TO ASSEMBLE YOUR LANSINOH® MANUAL BREAST PUMP

NOTE: Prior to assembling your Lansinoh® Manual Breast Pump, ensure that all parts have been cleaned and sterilized as detailed in section 2 of this manual.

- 1 Wash your hands thoroughly.

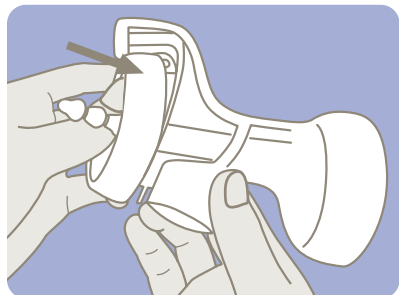- 2 Insert the stem into the silicone diaphragm.


- 3 Place the silicone diaphragm and stem on top of the flange body. Make sure it fits securely around the rim by pressing down with your fingers to ensure a perfect seal.*


*These parts are designed to fit snugly. For easier assembly, just dampen the rim of the silicone diaphragm with water

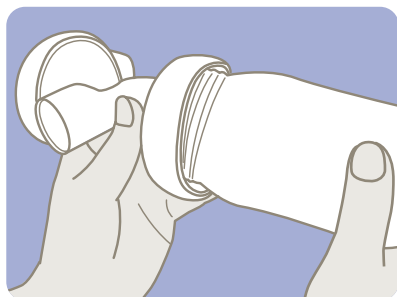- 4 Gently secure the white valve onto the flange body from underneath, making sure the white valve is flush with the flange body. It is not necessary to use excess force when attaching the white valve. *If you find that the white valve is difficult to remove, you have pushed it too far onto the breast flange body.*

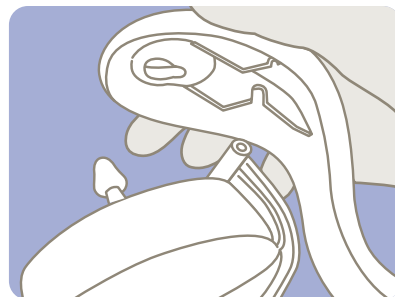
- 5 Screw the bottle onto the flange body.


- 6 Align the groove in the handle with the flange body and gently press the handle down. Push until you hear it click.

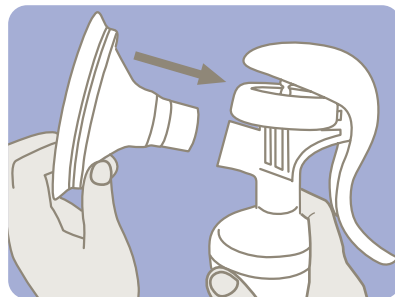
- 7 Adjust the stem and handle so that the stem fits the first notch of the handle, known as the Let-Down Mode setting.**

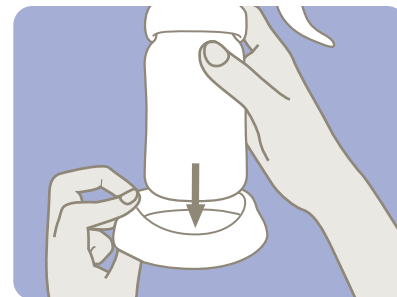- 8 Select the flange that fits best. Attach the ComfortFit™ Flange to the flange body.

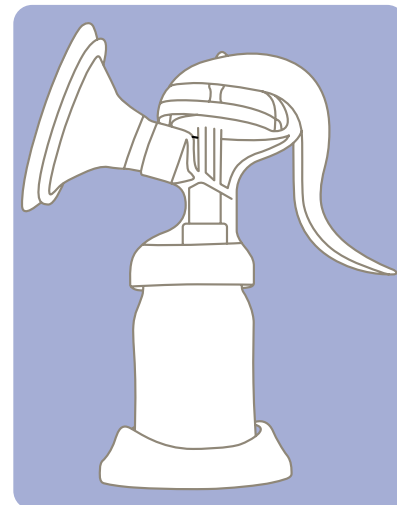- 9 To make the pump completely stable, insert the bottle into the bottle stand.

- 10 When fully assembled, this is what your pump should look like.

**This breast pump has two modes, Let-Down Mode and Expression Mode. You can easily switch between the two modes by adjusting the position of the handle of the stem.

4. HOW TO USE YOUR LANSINOH® MANUAL BREAST PUMP

If possible, wait to begin pumping until your breastmilk supply and breastfeeding schedule are well established. This is normally at least 3-4 weeks after birth, unless told otherwise by your doctor or breastfeeding advisor.

- 1 Wash your hands thoroughly.
- 2 Make sure you have sterilized and carefully assembled your pump exactly as described in the sections “How to Clean and Sterilize Your Lansinoh® Manual Breast Pump” and “How to Assemble Your Lansinoh® Manual Breast Pump.”
- 3 Center the pump opening over your nipple and gently press it against your breast ensuring that your nipple is centered in the tunnel of the flange. Press the ComfortFit™ Flange of the pump firmly against your breast making sure the seal is secure so that no air can escape. You can create a good seal by using your other hand to support your breast from underneath.

If your nipple is unable to be centered within the tunnel of the flange without rubbing, you may need the larger size ComfortFit™ flange included in the pack.

- 4 As you gently push down on the pump handle you will feel the suction on your breast. You do not need to depress the handle fully to create suction, only as much as is comfortable.
- 5 You can create gentle suction pressure in preparation for expressing by setting the handle in the first (top) notch of the stem. Confirm that your nipple is moving and squeeze and release the handle until you find your preferred pace.

Warning: Do not continue pumping for more than five minutes at a time if no result is achieved. Try expressing at another time during the day.

Please remember: Should pumping become very painful or uncomfortable, you should stop using the pump and consult your doctor or breastfeeding advisor.

USING LANSINOH'S 2-PHASE TECHNOLOGY

This breast pump has two modes, Let-Down Mode and Expression Mode. You can easily switch between the two modes by adjusting the position of the handle on the stem.

Let-Down Mode

This mode gently stimulates the breast. It is designed to mimic the way a baby feeds at the beginning of a feeding. It is one of the features that encourages milk let-down and encourages your milk to begin to flow. You can create gentle suction pressure by setting the handle in the first (top) notch in the stem and using quick pulses with the handle.

Expression Mode

This mode allows you to express breastmilk from the breast. Switch to this mode by setting the handle in the second (bottom) notch in the stem. To get the strongest suction fully depress the handle. For less suction strength depress the handle less.

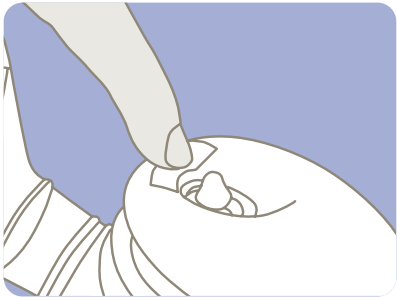
USING LANSINOH'S 2-PHASE TECHNOLOGY, CONTINUED


To start with the Let-Down Mode, set the stem in the first (top) notch of the handle and squeeze the handle. Your breastmilk will soon start flowing even though you may not use all the suction the pump can generate.

Once breastmilk is flowing (1 – 2 minutes) change to Expression Mode.

To switch to Expression Mode, gently press the “PUSH” part at the tip of the handle to set the handle to the second (bottom) notch of the stem.

Continue to squeeze and release the handle to find the appropriate suction pressure. If the suction pressure feels weak when expressing your breastmilk, squeeze the handle all the way. The suction pressure changes based on how hard you squeeze the handle. Expressing in comfort is more important than setting a strong suction pressure. Please express milk at a suction pressure that feels appropriate for you.

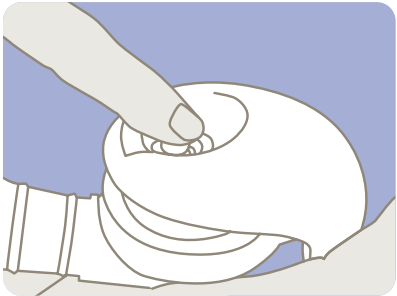*To return to Let-Down Mode from Expression Mode, (1) Pull the tip of the stem towards you, (2) gently squeeze the handle, and set it in the first (top) notch.

5. BREASTMILK STORAGE GUIDELINES

Breastmilk Storage and Thawing Guidelines For Healthy Full-Term Babies

Check with your healthcare provider or an International Board Certified Lactation Consultant for specific storage instructions. When you are freezing your breastmilk, do not fill containers more than ¾ full to allow for expansion from freezing. Date containers with the date you expressed your milk for future reference. When freezing, store in the back of the freezer and do not store milk in the door as temperature can fluctuate as door is opened and closed.

WHERE	TEMPERATURE	TIME
At Room Temperature	66-78°F (19-26°C)	4 hours (ideal) up to 6 hours (acceptable)
In a Refrigerator	<39°F (<4°C)	72 hours (ideal) up to 8 days (acceptable)
Freezer	0°F (-18°C)	3 months (ideal) up to 6 months (acceptable)
Deep Freezer	-4°F (-20°C)	6 months (ideal) up to 12 months (acceptable)

Source: La Leche League International 2014

Date all breastmilk and use the oldest breastmilk first.

Thawing

Thaw frozen milk in the refrigerator or place the containers under cool running water, gradually warming the water until the milk is thawed. Thawed milk can be kept in a refrigerator for up to 24 hours.

NOTE: DO NOT refreeze milk once it is thawed.

Gently swirl thawed milk before feeding to baby to mix the layers that have separated. Breastmilk is not homogenized and the “cream” will rise to the top of the bottle. Breastmilk can vary in color and consistency depending upon the time of the day that the milk was expressed, what you have eaten, and the age of the baby at the time of pumping.

NOTE: DO NOT MICROWAVE HUMAN MILK! Microwaving alters the composition of the milk. Microwaving also may create “hot spots” in the milk that could burn the infant.

NOTE: DO NOT thaw breastmilk in hot or boiling water.

6. TIPS ABOUT PUMPING

- Familiarize yourself with the pump and parts prior to use.
- The most important part of successful pumping is the let-down reflex. This is when your breastmilk begins to flow freely. Warm wet compresses applied to the breasts and gentle, circular massaging of the breasts prior to and/or during a pumping session can help encourage this natural reflex.
- Try to choose a time to pump when you are not rushed and will not be interrupted. Trying to relax and thinking of your baby or looking at a photo of your baby can also be helpful when pumping your breastmilk.
- Overall a pumping session can last about 20-30 minutes but it depends on your body. Remember, successful pumping is a learned art. Early practice sessions may be shorter or longer than stated here and may result in a small amount of breastmilk collected, but that does not mean your body is not making enough milk or that you are doing anything wrong. Milk is produced on a supply and demand basis and pumping your breasts regularly in addition to nursing your baby stimulates and maintains your breastmilk supply.
- If breastfeeding problems occur, call your physician, an International Board Certified Lactation Consultant, or breastfeeding counselor for an in-person consultation.
- You may pump one breast while nursing the baby on the other. This allows you to maximize pumping effectiveness using the let-down reflex stimulated by your baby's suckling.
- Make sure the tunnel of the breast flange is tilted slightly downward to allow your breastmilk to flow naturally toward the bottle.

Precautions: When feeding expressed breastmilk:

- DO NOT MICROWAVE BREASTMILK! Microwaving alters the composition of the milk and may also create "hot spots" in the milk that could burn the infant.
- Always test the temperature of any liquid before feeding your baby.
- Do not let a child take a bottle to bed or self feed.
- To prevent possible choking hazard, test strength of the nipple by pulling on the bulb portion. Discontinue use if any tear or crack appears.

7. TROUBLESHOOTING FOR THE LANSINOH® MANUAL BREAST PUMP

Lack of Suction / Decreased Suction

If it feels like the suction has decreased, try the following:

- Make sure that all pump parts have been properly dried and properly assembled.
- Inspect the white valve for a tear or pinhole. This component is critical to achieving proper suction. If necessary, replace the white valve with the spare valve provided with the pump.
- Ensure that the silicone diaphragm and stem is fitted securely to the pump body and a perfect seal is created.
- Ensure that the ComfortFit™ Flange is fitted securely to the flange body and a perfect seal is created.
- If your nipple is unable to be centered within the tunnel of the flange without rubbing, you may need the larger size ComfortFit™ flange included in the pack.

No Breastmilk Expressed

If you are not successful in expressing any milk, try the following:

- Make sure that all the parts have been properly dried and assembled and that proper suction is created.
- Try to relax as successful pumping is a learned art and may take several practice attempts to get started. Gentle breast massage before and during pumping can also aid in milk let-down and flow.
- Please refer to the "Tips About Pumping" section.

Pain When Pumping

If you are experiencing pain when expressing breastmilk, try the following:

- You may be pumping too hard. It may not be necessary to use all of the suction that the pump can generate. Try depressing the handle half way.
- You may need a larger or smaller flange. Try the other flange included in this package. Consult your lactation consultant for an assessment and sizing.
- Consult your physician, breastfeeding advisor or lactation consultant.

Lost or Broken Parts

Spare parts are available through Lansinoh® Customer Service at 800-292-4794 or on our website at www.lansinoh.com/store.

If you are still experiencing problems, please contact Lansinoh® Customer Service at **800-292-4794** or contact your physician or breastfeeding advisor.

Manufacturer: Lansinoh Laboratories Saglik Gerecleri Tasarim San. Tic. Ltd. Sti.
Zafer SB Mah. Gunduz Sk. No:13 E.S.B Gaziemir/ Izmir/ Turkiye

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This product is a personal care product and therefore cannot be returned once opened. If you experience any problems with this product, please telephone our toll free customer service line at 800-292-4794. Our representatives will be happy to assist you.

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Lansinoh breastfeeding products are manufactured and distributed by Lansinoh Laboratories, Inc in the US.

©2016 Lansinoh Laboratories, Inc. All rights reserved. Printed in Turkey
MN50520IF0616

Distribuido por Lansinoh Laboratories, Inc
Alexandria, VA 22314 USA
800-292-4794
Lansinoh.com

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Fabricante: Lansinoh Laboratories Saglik Gerecleri Tasarim San. Tic. Ltd. Sti.
Distributed by Lansinoh Laboratories, Inc
Lansinoh® es una marca registrada de Lansinoh Laboratories, Inc.
©2016 Lansinoh Laboratories, Inc. Todos los derechos reservados. Impreso en Turquía.