Setup Instructions

Instructions d'installation | Instrucciones de configuración

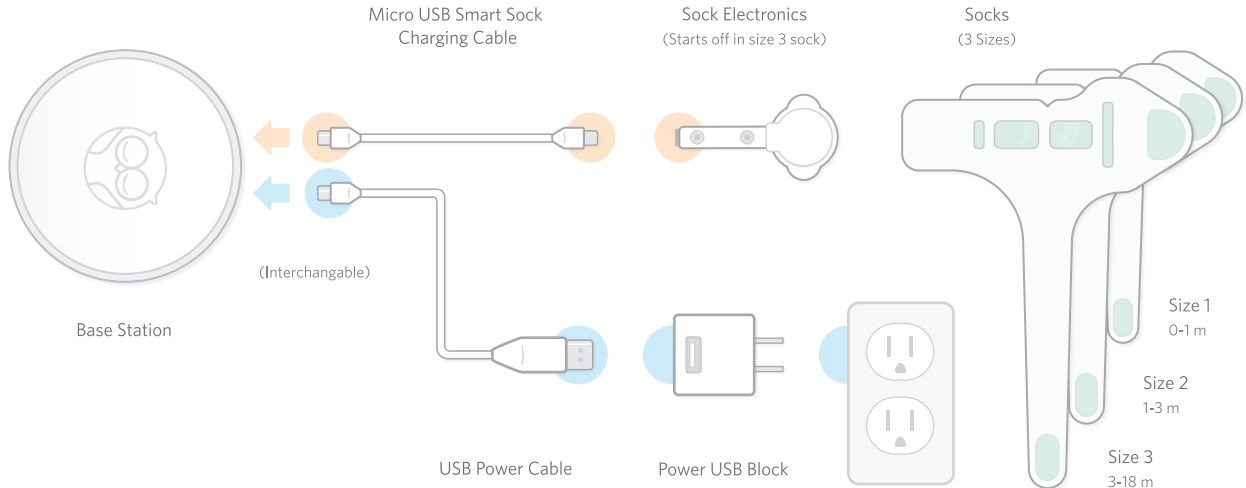
Let's get Started

1

Plug the Sock Electronics into the Base Station using the Micro USB.


2

Plug the Base Station into an electrical socket. The Smart Sock will now start charging.

3

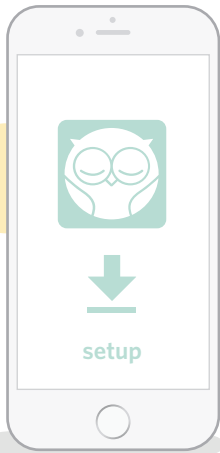
Download the Owlet App to set up the Owlet Smart Sock™ and Base Station.

The Owlet App

4

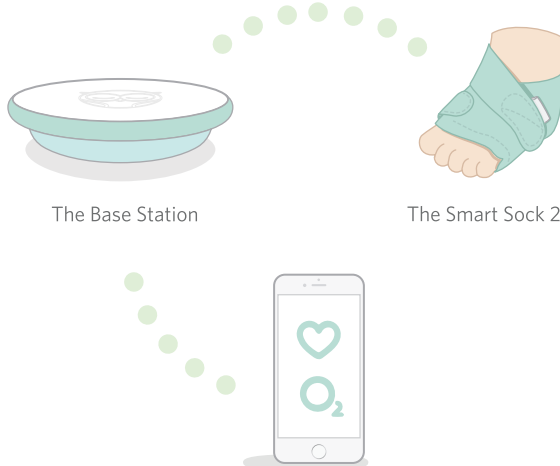Rest assured with Owlet's help to watch over your baby. Use the Base Station as the primary notification system and the App as your secondary.

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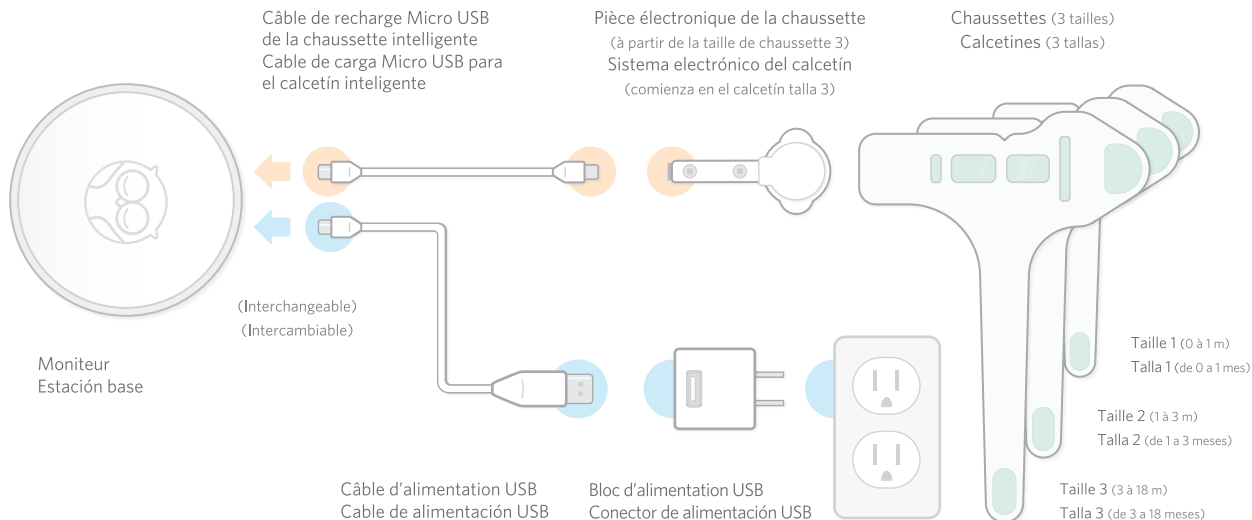
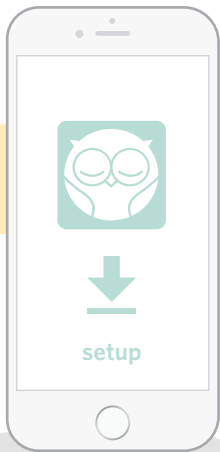
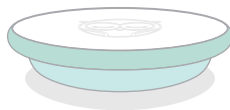
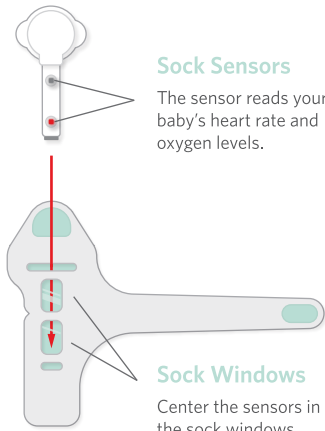
L'application Owlet
La aplicación de Owlet

Smart Sock Quick Reference

The Owlet Smart Sock™ 2 uses two sensors to track your baby's readings.

Interchangeable Sock

The Owlet sock can go on either the **right** or **left** foot. For proper placement on either foot, place the **notch** and **sock sensor** on the outside of the foot behind the pinky toe.

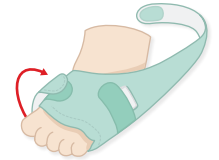
Right Foot

Dome on top of foot

Left Foot

Dome on bottom of foot

Sock Fit

Don't overtighten. If the strap goes past the velcro, try going down a size.

Base Station Quick Reference

See your baby's status with a glance at the Base Station

Sock Light

Green light indicates sock is connected to Base Station.

Registration

Push once when prompted during setup in the App.

WiFi Light

Green light indicate Base Station is connected to wifi.

On / Off

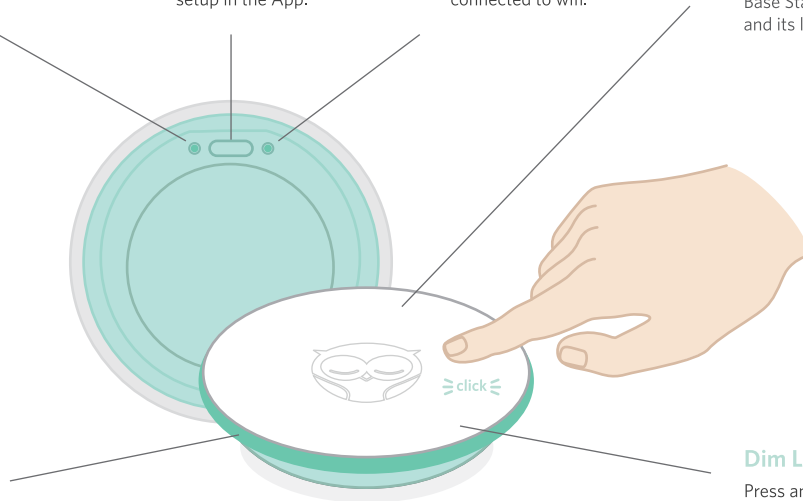
Press and release to turn off Base Station. The Base Station will beep, and its light will turn off.

Status Light

Light ring lets you know the status of your baby. More information is available in the App.

Dim Lights

Press and hold to switch between bright daytime mode to dim nighttime mode.


Base Station Color Reference


The Base Station has 5 different color status and notifications. To hear examples, go to the App.

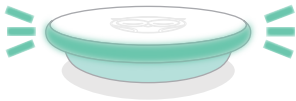Status


Blinking green light

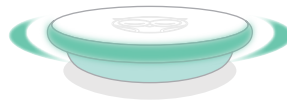Getting first heart rate and oxygen readings

Slow pulsing green light

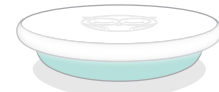
Heart rate and oxygen levels are in range

White static light


The Smart Sock is plugged in and charging

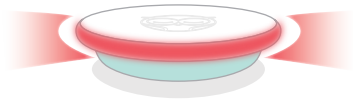Notifications


Red flashing light 🔊

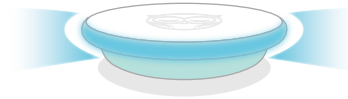Heart rate or oxygen levels are outside preset zone

Blue flashing light 🔊

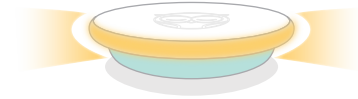The Smart Sock is disconnected from the Base Station

Yellow flashing light 🔊

The Smart Sock fell off or is poorly placed, or shifted from proper placement

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Guide de référence | Referencia rápida

Troubleshooting Guide

Initial Setup | Notifications | Connectivity

Initial Setup

Problem	Causes	Solutions
<ul style="list-style-type: none"> Cannot see the Smart Sock 2 Wi-Fi during setup 	<ul style="list-style-type: none"> The Base Station is not turned on Already connected to Smart Sock 2 Wi-Fi The Base Station is out of range 	<ul style="list-style-type: none"> Plug in the Base Station Check if the Wi-Fi light on the bottom of the Base Station is on. If so, you can skip this step and continue setting up your account in the app Move the Base Station closer to your Wi-Fi router
<ul style="list-style-type: none"> Cannot see your home Wi-Fi in the Owlet App 	<ul style="list-style-type: none"> Your home router is 5.0 GHz (Owlet only supports 2.4 GHz) You may have a hidden router 	<ul style="list-style-type: none"> Make sure your iOS device is connected to a 2.4 GHz network (most 5.0 GHz routers also support 2.4 GHz) Visit owletcare.com/support for instructions on hidden routers
<ul style="list-style-type: none"> The Base Station will not connect to your home Wi-Fi 	<ul style="list-style-type: none"> Password is incorrect The Base Station is out of range of your Wi-Fi router Your home Wi-Fi is not working 	<ul style="list-style-type: none"> Double check your internet password Move the Base Station closer to your internet router for setup, then move the Base Station back to the nursery after set up Try restarting the phone and reconnecting to your Wi-Fi Reset your Wi-Fi router
<ul style="list-style-type: none"> The Base Station remains white after the Smart Sock is unplugged 	<ul style="list-style-type: none"> Software update is in progress 	<ul style="list-style-type: none"> Plug the Smart Sock back into the Base Station until the update is complete
<ul style="list-style-type: none"> The Base Station will not pair to your Owlet account 	<ul style="list-style-type: none"> The Base Station and your iOS device are not on the same Wi-Fi network Holding the small Register Button 	<ul style="list-style-type: none"> Verify that your iOS device is connected to same Wi-Fi network as the Base Station Press and release (do not hold) the small Register Button on the bottom of the Base Station and wait a few seconds before trying to pair your Owlet account again Connect your iOS device to the same 2.4 GHz Wi-Fi network as your Base Station

*Need Technical Support? Email us at contact@owletcare.com or call us at +1 (844) 334-5330 from 7AM - 9PM (MST) Mon. - Sat.
For all general questions chat with us at owletcare.com/chat

Notifications

Problem	Causes	Solutions
<ul style="list-style-type: none"> The Base Station is flashing blue and making noise 	<ul style="list-style-type: none"> The Base Station is out of range of the Smart Sock The caretaker's body is blocking the Smart Sock to Base Station connection. This is common during feedings, diaper changes, snuggles, etc. Low/dead battery 	<ul style="list-style-type: none"> Move the Base Station closer to the Smart Sock Reposition your baby or consider turning off monitoring during feedings, diaper changes, snuggles, etc. Recharge the Smart Sock
<ul style="list-style-type: none"> The Base Station is flashing red and making noise 	<ul style="list-style-type: none"> Low/high heart rate Low oxygen levels The Smart Sock does not fit properly 	<ul style="list-style-type: none"> Make sure the notch is on the outside of your baby's foot just 1-2mm behind the pinky toe. Make sure the Smart Sock is snug (if not, it is difficult to get a reading. False notifications could occur) Make sure the sensors are centered in the windows of the Fabric Sock Consider using a different Fabric Sock size
<ul style="list-style-type: none"> The Base Station is flashing yellow and making noise 	<ul style="list-style-type: none"> Difficulty getting a reading The Smart Sock was placed incorrectly The lighting in the room is too bright 	<ul style="list-style-type: none"> Make sure the notch is on the outside of your baby's foot just 1-2mm behind the pinky toe. Make sure the Smart Sock is snug (if not, it is difficult to get a reading, and more false alerts will occur) Make sure the sensors are centered in the windows of the Fabric Sock Dim the lighting in your baby's room

Connectivity

Problem	Causes	Solutions
<ul style="list-style-type: none"> The Base Station is disconnected from the internet 	<ul style="list-style-type: none"> The Base Station is out of range of your Wi-Fi router Your home Wi-Fi is not working 	<ul style="list-style-type: none"> Move the Base Station and the Wi-Fi router closer together (neither Wi-Fi range extenders nor second routers will help) Reset your Wi-Fi router
<ul style="list-style-type: none"> Unable to charge the Smart Sock 	<ul style="list-style-type: none"> The Smart Sock is not plugged in all the way The Smart Sock requires a reset 	<ul style="list-style-type: none"> Plug the Smart Sock in to charge until you hear a click and it is blinking green Turn the Sock Electronics off, then back on.
<ul style="list-style-type: none"> Seeing dashes instead of heart rate and oxygen 	<ul style="list-style-type: none"> The Smart Sock is trying to get a reliable reading Too much movement Poor Smart Sock placement Poor Fabric Sock fit 	<ul style="list-style-type: none"> Make sure the sensors are centered in the windows of the Fabric Sock Do not use rockers or vibrating pads, as they may interfere with the proper functioning of the device. Check the Smart Sock placement Check the Fabric Sock size

Tips for a Safe Sleep Environment

Some friendly tips to keep your baby healthy and safe.

- Always place your baby alone and on their back to sleep. Placing your baby to sleep on their side or on their stomach is not safe.
- Use a crib that meets current safety standards. Use a firm mattress that fits snugly in the crib and is covered with only a tight-fitting crib sheet.
- Do not put anything soft, loose, or fluffy in your baby's sleep space. This includes pillows, blankets, comforters, bumper pads, stuffed animals, toys, or any other soft items.
- Use a wearable blanket, or similar type sleeper, instead of traditional blankets to keep your baby safe and warm.
- Room-share instead of co-sleeping. Keep your baby's sleep space separate from, but close to, your bed. This will keep them safe, and make feeding easier. The Smart Sock does not function properly with co-sleeping. Your body will interfere with the Smart Sock's wireless signal.
- Avoid falling asleep with your baby in your bed, couch, or armchair.
- Never allow your baby to sleep on any soft surfaces such as adult or child-sized beds, sofas, chairs, waterbeds, pillows, cushions, comforters, sheepskins, or anything similar.
- Do not use pillows, wedges, or positioners to prop your baby up, keep them on their back, or prevent them from rolling.
- Make sure your baby doesn't get too warm. Use light sleepwear on your baby and keep the room temperature comfortable for a lightly-clothed adult.
- Consider offering a pacifier to your baby at naptime and bedtime. For breastfed babies, delay introducing a pacifier until 3-4 weeks of age, when nursing is well-established.
- Educate everyone who cares for your baby about these rules for safe sleep!